

Innovative Technology
For Your Medical Aesthetic Needs

HYDRA@BEAUTYTM

The Professional skin care system
for perfect skin



SPECIFICATION

Applicators	Aqua cleansing Electroporation Diamond peeling
Input Power	100-132V / 200-240 V~, 50/60 Hz
Panel	7" Inch
Weight	45Kg
Dimension (WxDxHmm)	344 x 435 x 1060



Exclusively distributed in Benelux by:
Medi-Invest Nv - Georges Wittouckstraat 20 - 1600 Sint-Pieters-Leeuw - BELGIUM
www.Medi-Invest.eu - info@medi-invest.eu - Tel: +32 (0)2 381 27 00



HYDRA@BEAUTY™

The combination of 3 functions for complete skin care


HydraBeauty is a skin care system which uses its specialized tip to exfoliate, pore cleanse, and also penetrate the differentiated premium solution into skin by using the principle of electroporation that it will make the skin clear and clean.

Convenient multi-function

- ▶ One step Clean System
- ▶ Multi-polar Technique that does not require a ground
- ▶ Face & Body care System

A safe and effective function

- ▶ 5 level adjustable Vacuum
- ▶ In flow rate in 3 levels
- ▶ 10 level adjustable electroporation

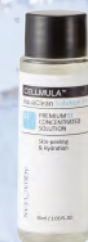


Solution 1

Remove sebum and oxidized waste deeper in the pores by addition of Lactic Acid.



Solution 2

Dissolve oxidized dead skin and calms ACNE, rash, and other skin problem by addition of Salicylic Acid.

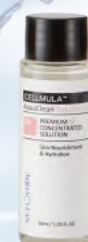

Solution 3

Supplement nutrients and deep moisturizing on the cleaner skin by addition of Portulaca Ieracea extract.



Solution 4

Containing ethanol, solution 4 disinfects and cleanses the entire system.



Dermal Waste Removal



Various Solution Can be Applied




Skin Cleaning

1 Aqua Cleansing system



Radial Tip

Solution is emitted from the central aperture and spread along the solution path while in contact with the skin, dissolving impurities then exits through the waste inlet port.

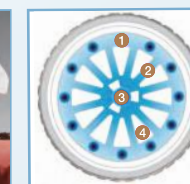

- 1 Waste inlet port
- 2 Solution path
- 3 Solution emission
- 4 Skin contact region


2 Solution penetrates through electroporation



- Effective solution penetration for skin rejuvenation and nutrition
- 3 types of LED light ● Red ● Blue ● Red & Blue

Principle

It is a method of making substances of certain molecular weight going through the cell membrane. Electropulses disturb phospholipid bilayer to temporarily make pores on skin surface and they are closed when the electropulses stop and the solution penetrates into the cell through the pores.

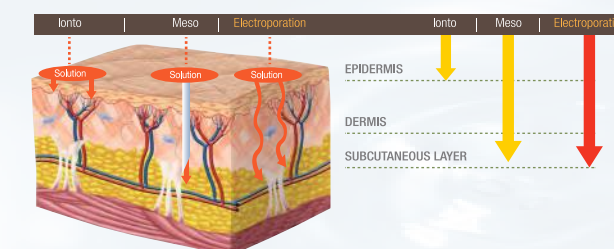

3 Peeling through Diamond tips



Diamond Peeling Tips

Exfoliates dead skin and impurities by vacuum pressure which maximizes solution absorption effect resulting in much smoother skin and tone. It offers 3 types of tips for different skin types and applied regions.



ES-10



ES-20



ES-30